

## INSTRUCTIONS OF USE FOR COVERALL



**【Specification】** Coverall

**【Material】** The product adopts "PP + PE" (polypropylene + polyethylene breathable film) composite material, which is composed of hat, jacket and pants.

**【Scope】** This product is for general protection.

**【Instructions】** The method of wearing one-piece protective clothing is: unfold the protective clothing, open the zipper, extend into the legs from the opening of the zipper, then wear the top, hand out from the sleeve, wear a hood, and pull the zipper from the front of the chest. And buckle in the reverse direction, put on the shoe cover.

### **【Notes】**

1. Protective clothing is limited to one-time use, and it should be discarded after use.
2. Do not use if there is any breakage or any improper part of the zipper, stitching, etc.
3. Protective clothing should avoid prolonged contact with chemicals.
4. This product is a non-sterile, non-flame retardant product.

**【Storage】** Storage in dry, and ventilated environment with no corrosive gas, avoid high temperature, humidity and long hours of direct sunlight, keep away from open flame.

**【Production date】** See certificate of qualification

**【Manufacturer】** Sword Xiantao Disposable Protective Products Factory

**【Address】** Yehe Village, Liukou Industrial Zone, 433000 Xiantao City, Hubei Province, China



Warning: Non-sterile product    One-time use